



# ANTI-STATIC SILICONE FACE MASK

## CLEANING & INSTRUCTIONS FOR USE



### AVAILABLE IN 3 SIZES:

Small  
silicone  
mask



Medium  
silicone  
mask



Large  
silicone  
mask



Medical Developments International Limited ABN 14 106 340 667  
4 Caribbean Drive, Scoresby, Victoria 3179, Australia  
Telephone +61 3 9547 1888 Fax +61 3 9547 0262  
Email mdi@medicaldev.com Web www.medicaldev.com

LI-BATAS-MASK-V1

## CLEANING INSTRUCTIONS

<p>1</p>	<p>2a</p>	<p>2b</p>	<p>3</p>	
<p>Rinse the mask in clean warm water.</p>	<p>Add a mild detergent to the water and wash the mask. Rinse the mask thoroughly in clean water to remove detergent.</p>	<p>Place on top rack of dishwasher with interior side facing up as in picture. Use dishwasher detergent with normal cycle. Avoid washing with overly dirty dishes.</p>	<p>Shake out excess water and allow to air dry.</p>	<p><b>STORAGE:</b> Store away from direct sunlight.</p> <p><b>DISPOSAL:</b> There are no special disposal instructions for the Anti-static Silicone Face Mask.</p> <p><b>REPLACE:</b> The Anti-static Silicone Face Mask should be replaced after 12 months.</p>

## INSTRUCTIONS FOR USE

<p><b>WARNINGS</b> For reuse by a single patient. This mask is designed to mate with a standard 22 mm diameter fitting. Ensure connections are secure at all times. If you experience skin irritation consult your doctor.</p>	<p>1</p> <p>Remove mask from box/bag.</p>	<p>2</p> <p>Connect mask to breathing device.</p>	<p>3</p> <p>Ensure connection is secure.</p>	<p>4</p> <p>Apply mask to face, covering nose and mouth and check for effective seal.</p>	
<p><b>WARNINGS</b> For reuse by a single patient. This mask is designed to mate with a standard 22 mm diameter fitting. Ensure connections are secure at all times. If you experience skin irritation consult your doctor.</p>	<p>Remove mask from box/bag.</p>	<p>Connect mask to breathing device.</p>	<p>Ensure connection is secure.</p>	<p>Apply mask to face, covering nose and mouth and check for effective seal.</p>	<p><b>NOTE:</b> If there is any concern of mask failure, replace mask immediately. Anti-static Silicone Face Mask is not made with natural rubber latex.</p>

LOT