




## ANTI-STATIC COLLAPSIBLE SPACER SINGLE PATIENT USE

### INSTRUCTIONS FOR USE AND CLEANING

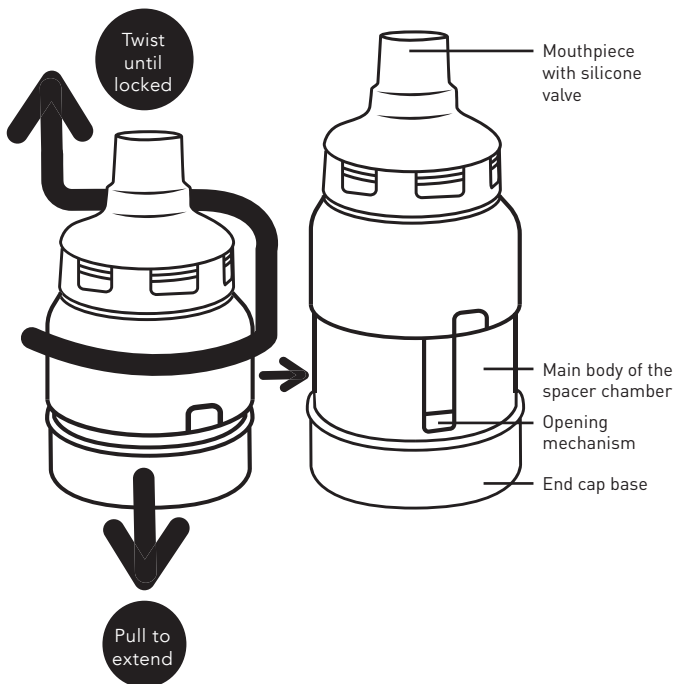

To extend, pull both ends of the spacer, then twist the top section until you hear a click. This will create a secure, locked spacer.



Medical Developments International Limited  
ABN 14 106 340 667

4 Caribbean Drive, Scoresby Victoria 3179, Australia  
Telephone +61 3 9547 1888 Fax +61 3 9547 0262

Email [mdi@medicaldev.com](mailto:mdi@medicaldev.com)  
Web [www.medicaldev.com](http://www.medicaldev.com)

9042MDI LI-BAT-AS-COL-V1

# INSTRUCTIONS FOR USE

**PLEASE READ AND FOLLOW ALL THE CONTENTS OF THIS LEAFLET BEFORE USING YOUR DEVICE**

The Breath-A-Tech® Anti-static Collapsible Spacer is a hand-held, portable valved holding chamber (VHC). The spacer is placed between a pressurised metered dose inhaler (pMDI, or 'puffer') and the patient's mouth, to function as a reservoir from which aerosol medication is dispensed, in order to minimise the delivery of large aerosolised particles. By holding the medication after aerosolisation, the spacer reduces the direct delivery of large aerosolised particles into the mouth, which are intended for the lower airways. Suitable for adults and children to use with most standard puffers in the treatment of ASTHMA or COPD. The device is for single patient use only. It can be used in most environments, including a patient's home, hospitals, emergency services and clinics. Not made with natural rubber latex or BPA.

<p><b>⚠ WARNINGS</b></p> <p>Ensure Instructions for Use have been read prior to use and are kept available at all times.</p> <p>Always follow the instructions provided with your puffer. Ensure the spacer is fully extended and locked into position prior to use.</p> <p>Children must be supervised by an adult or caregiver during use.</p> <p>If breathing problems or symptoms persist, contact your physician and seek prompt medical attention.</p> <p><b>REPLACE WITHIN 12 MONTHS OF FIRST USE</b></p> <p>Note: The Breath-A-Tech® Anti-static Collapsible Spacer may be disposed of in general waste.</p>	<p><b>1</b></p> <p>Pull both ends of the spacer, then twist the top section until you hear a click. Ensure the extension of the spacer is secure before use.</p>	<p><b>2</b></p> <p>⚠ Before use check for any foreign objects, broken or deteriorated parts. Replace the Breath-A-Tech® Antistatic Collapsible Spacer immediately if any damage or missing parts are evident.</p>	<p><b>3</b></p> <p>Prepare the puffer by following the instructions provided with your medication.</p>	<p><b>4</b></p> <p>Ensure the cap from the puffer has been removed. Push the puffer into the end cap base of the Breath-A-Tech® Anti-static Collapsible Spacer.</p>	<p><b>5</b></p> <p>Put the spacer mouthpiece between your lips and make an airtight seal. Depress your puffer and you will see a fine spray inside the Breath-A-Tech® Anti-static Collapsible Spacer. Breathe in and out through the spacer for 4-6 breaths or follow the instructions provided with your puffer, or as directed by your healthcare professional. Always follow the instructions provided with the puffer or your asthma/COPD action plan on how long to wait before repeating this step. Only use one dose from the puffer at a time.</p>	<p><b>6</b></p> <p>After use, close the extension of the Breath-A-Tech® Anti-static Collapsible Spacer by holding both ends of the spacer and twisting the top section until you hear a click. Push both ends of the spacer together to collapse. Store in a clean, dry place, away from direct sunlight.</p>
--	--	---	--	---	--	---

# CLEANING INSTRUCTIONS

	<p><b>DISHWASHING METHOD</b></p>	<p>OR</p> <p><b>HANDWASHING METHOD</b></p>	<p><b>1</b></p>	<p><b>2</b></p>	<p><b>3</b></p>	
<p><b>DISASSEMBLING YOUR SPACER FOR CLEANING</b></p> <p>To extend the device, pull both ends of the spacer, then twist the top section until you hear a click. Remove the end cap base by placing your thumb in the hole, and pulling the end cap simultaneously. ⚠ Do not remove the mouthpiece and silicone valve.</p>	<p>The Breath-A-Tech Anti-Static Collapsible Spacer should be cleaned weekly in a dishwasher, up to 70°C. Place the end cap base and the spacer on the top shelf of the dishwasher. Position the end cap base flat. Place the spacer in an upright position in the dishwasher, with the mouthpiece facing up.</p>	<p>The Breath-A-Tech® Anti-static Collapsible Spacer should be cleaned at least once a week. ⚠ Do not use a brush or an abrasive sponge to clean the Breath-A-Tech® Anti-static Collapsible Spacer, particularly near the valve area, as this can damage your spacer.</p>	<p>Wash both parts in warm water with a dishwashing liquid</p>	<p>Rinse both parts in clean water.</p>	<p>Shake off any excess water and allow both parts to air dry in a vertical position, with the mouthpiece facing up.</p>	<p><b>ASSEMBLING YOUR SPACER AFTER CLEANING</b></p> <p>When both parts of the Breath-A-Tech® Anti-static Collapsible Spacer are completely dry, firmly replace the end cap base by pushing onto the spacer. Close the extension of the spacer by holding both ends of the spacer, then twist the top section until you hear a click. Push both ends of the spacer together to collapse.</p>

<p><b>Symbols</b></p>	<p>Manufacturer</p>	<p><b>LOT</b></p> <p>Batch code</p>	<p><b>REF</b></p> <p>Catalogue number</p>	<p>Consult instructions for use</p>	<p>Caution</p>	<p>Use-by date</p>
-----------------------	---------------------	-------------------------------------	---	-------------------------------------	----------------	--------------------

If a replacement leaflet is required, please contact Medical Developments International Limited